

Help keep the baby warm and safe

Never walk away with a wall down or door open



Scan this OR code to watch a short 5-minute video on opening and closing porthole doors and lowering/raising walls

gehealthcare.com

time the bedside panel or porthole is closed to make sure the porthole door is latched.





LATCHED



bedside panel every time the panel is closed to make sure the panel is latched.

You must pull on the

The red tab shows the latch is **not engaged.**

You must pull on the bedside panel every time the panel is closed to make sure the panel is latched.

The red tab shows the latch is **not engaged.**



LATCHED



UNLATCHED